### ANKLE REMOVAL

### (ACCESS TO REAR BUMPER, ANKLE BUSHINGS AND MIDSTANCE PAD)

- 1. Use the Foot Horn to don/doff the foot shell. Remove the CPI Sock and replace as needed.
- 2. Use a 6 mm and 4 mm hex keys to remove the axial pin screw.
- 3. Attach the pin guide to the axial pin. Push and rotate clockwise.
- 4. Unscrew the axial pin from the pin guide and remove it from the ankle bone.
- 5. Remove the ankle bone to access the rear bumpers, ankle bushings and midstance pad.
- 6. For re-assembly, lubricate the axial pin, inside and outside of the ankle bushings, ankle bushing pocket and the contact surface of the midstance pad, then reverse steps 1-4. Torque the axial pin screw to 4 N·m (36 in·lbs).

### DO NOT lubricate front and rear bumpers.

### FOREFOOT REMOVAL

#### (ACCESS TO FRONT BUMPER AND LOWER BUSHINGS)

- 1. To remove the PreLoad Insert, position the PreLoader Tool on the foot to remove the load off of the insert. Tighten the handle down on the forefoot then swing the cam lever to either side. For detailed instructions refer to the Preloader Tool TechTip.
- 2. Remove the front bumper and axial pin as instructed (above) for the Ankle Bone.
- Remove the forefoot bone to access the lower bushings.
- 4. For re-assembly, lubricate the axial pin, inside and outside of the lower bushings, then reverse steps 1-4. Torque the axial pin screw to 4 N·m (36 in lbs)

### **GAIT MATCHING® GUIDELINES**

Contact College Park Technical Service if you have questions about modifying the gait match.

### WARRANTY INSPECTION AND MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups according to the warranty Inspection schedule below.

High patient weight and/or impact level may require more frequent inspections. Soft component wear depends on the patient weight, impact level and environment. We recommend you inspect the following applicable parts for excessive wear and fatigue at each warranty inspection and replace as needed.

- Soft components (disassemble, inspect & re-lubricate) CPI Sock
- **Composites and Adapters**

Foot Shell

WARRANTY INSPECTION SCHEDULE FOR TRUSTEP: SIX MONTHS, THEN ANNUALLY.

#### **TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365**

College Park's regular office hours are Monday through Friday, 8:30 AM - 5:30 PM (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.



800.728.7950 | 586.294.7950 | college-park.com

EC REP EMERGO EUROPE

The Hague, Netherlands

COLLEGE PARK INDUSTRIES, INC 27955 College Park Dr. Warren, MI 48088 USA

©2018 College Park Industries, Inc. All rights reserved. College Park, Trustep and Gait Matching are registered trademarks of College Park Industries.

MADE IN THE USA 188 INS TS TIS 171108

Australian Sponsor EMERGO AUSTRALIA Prinsessegracht 20, 2514 AP Level 20, Tower II Darling Park 201 Sussex Street Sydney, NSW 2000 Australia







technical instructions



This diagram is to help familiarize you with the unique parts of the Trustep. These parts are referenced in the instructions and used when speaking with a technical service representative.

## PACKAGE CONTENTS

(1) Trustep Foot
(1) Foot Shell
(1) CPI Sock
(1) Sealing Boot (upon request)
(1) TruLube Lubricant
(1) Exo Block Kit (Exo Only)

## TOOLS RECOMMENDED

#2 Phillips Screwdriver Torque Wrench 4 mm and 6 mm Hex Keys Foot Horn Pin Guide Preloader Tool

## OPTIONAL

Exo Pyramid Tool Kit (Exo Only)

## KEY COMPONENTS

**OPTIONAL** 

14. Exo Block

**EXO MOUNT** 

13. Exo Pyramid Tool

15. Exo Ankle Bone

**16.** Exo Mounting Bolt

- 1. Ankle Bone
- 2. Heel Bone
- **3.** Forefoot Bone
- 4. Rear Bumper
- 5. Midstance Pad
- 6. Front Bumper
- 7. Ankle Bushings (2)
- 8. Lower Bushings (2)
- 9. Axial Pins (2)
- 10. Axial Pin Screws (2)
- **11.** PreLoad Insert
- 12. Insert Screw

# ▲ PROTECTIVE COVER ON DOME

Remove the protective cover on dome after alignment is completed and before patient leaves clinic.

## ENDOSKELETAL MOUNTING

Use only high quality proximal endoskeletal components (30 mm).

## EXOSKELETAL MOUNTING

- Remove Exo Ankle Bone, then attach to College Park Exo Block with anti-rotation pin(s) oriented. Apply Loctite® 242 to mounting bolt. Torque to 61 N·m (45 ft·lbs). To skip alignment and lamination, go to Step 9.
- 2. If using the alignable Exo option, attach Exo Pyramid Tool to Exo Block with four 6 mm screws. Torque to 15 N·m (11 ft·lbs).
- 3. Attach 30 mm endo components to the Exo Pyramid Tool and temporarily mount the socket.
- 4. Re-attach the ankle bone to foot, donning CPI Sock and foot shell, then perform a dynamic alignment.
- 5. Remove foot from Exo Block.
- 6. Mount aligned prosthesis in transfer jig. Lock socket and Exo Block in position.
- 7. Remove endo components and Exo Pyramid Tool.
- 8. Use desired method to span Exo Block to the socket and remove from jig. Shape and laminate to desired finish. Do not remove foam from the top of the Exo Block
- 9. Re-attach ankle bone to heel bone. Re-assemble foot, don CPI Sock and foot shell.

## DYNAMIC ADJUSTMENTS

DESIRED RESULT	ALIGNMENT CHANGE	COMPONENT CHANGE
Firmer Toe Response	Plantarflex the Trustep or move load line posterior	Front Bumper up one step firmer
Softer Toe Response	Dorsiflex the Trustep or move load line anterior	Front Bumper down one step softer or use MIN Insert
Firmer Heel Response	Dorsiflex the Trustep or move load line anterior	Rear Bumper up one step firmer
Softer Heel Response	Plantarflex the Trustep or move load line posterior	Rear Bumper down one step softer
Increase/Decrease Rotation, Inversion or Eversion	_	Change Ankle Bushings

## STATIC ALIGNMENT

For optimal function have the patient's weight balanced between the forefoot and heel. The load line divides the foot at 1/3 heel lever and 2/3 toe lever.

# 

- Do not expose this product to pH extremes or corrosive materials (water, salt water or other liquids).
- Failure to follow these technical instructions or use of this product outside the scope of its Limited Warranty may result in injury to the patient or damage to the product.
- Disassembly, modification or removal from the foot shell, or any further disassembly or modification of components will void the warranty.