



WARRANTY INSPECTION AND MAINTENANCE INFORMATION

College Park recommends that you schedule your patients for check-ups on the Horizon annually.

High patient weight and/or impact level may require more frequent inspections. Soft component wear depends on the patient weight, impact level and environment. We recommend you inspect the following applicable parts for excessive wear and fatigue at each warranty inspection and replace as needed.

- Composites and Adapters
- CPI Sock
- Foot Shell
- Heel Wedge (if applied)

TECHNICAL ASSISTANCE / EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday through Friday, 8:30 AM – 5:30 PM (EST). After hours, an emergency Technical Service number is available to contact a College Park representative.



**COLLEGE PARK
INDUSTRIES, INC.**
27955 College Park Dr.
Warren, MI 48088 USA

EMERGO EUROPE
Prinsessegracht 20, 2514 AP
The Hague, Netherlands


CE
MADE IN THE USA
529 INS HZLT TIS 171208

**Australian Sponsor
EMERGO AUSTRALIA**
Level 20, Tower II
Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia

©2018 College Park Industries, Inc. All rights reserved. College Park, Horizon, and Gait Matching are registered trademarks of College Park Industries.



*technical
instructions*

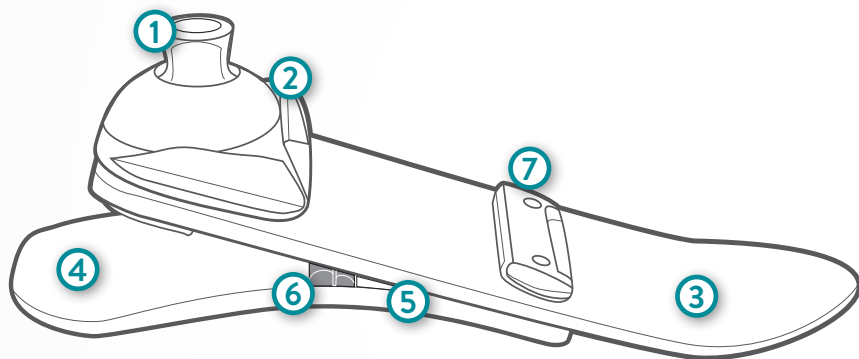

This diagram is to help familiarize you with the unique parts of the Horizon LT. These parts are referenced in the instructions and used when speaking with a technical service representative.

KEY COMPONENTS

1. Stainless Steel Pyramid
2. Aluminum Housing
3. Carbon Toe spring
4. Carbon Heel Spring
5. Permanent Wedge
6. Rounded Heel Wedge (Optional)
7. Nut Plate

PACKAGE CONTENTS

- (1) Horizon LT Foot
- (1) Foot Shell
- (1) CPI Sock
- (1) Horizon Wedge Kit

TOOLS REQUIRED

4mm Hex Key

TOOLS RECOMMENDED

Foot Horn

⚠ WARNING

- Do not expose this product to corrosive materials, salt water or pH extremes.
- Failure to follow these technical instructions or use of this product outside the scope of its Limited Warranty may result in injury to the patient or damage to the product.

⚠ PROTECTIVE COVER ON DOME

Remove the protective cover on dome after alignment is completed and before patient leaves clinic.

MOUNTING

Use only high quality proximal endoskeletal components.

ASSEMBLY AND DISASSEMBLY (for sock replacement)

Use the FootHorn to don and doff the foot shell. Remove the CPI Sock and replace as needed.

Any further disassembly or modification of components will void the warranty.

STATIC ALIGNMENT

For optimal function have the patient's weight balanced between the forefoot and heel. The load line divides the foot at 1/3 heel lever.

DYNAMIC ADJUSTMENTS

DESIRED RESULT	ALIGNMENT CHANGE	COMPONENT CHANGE
Firmer Toe Response	Plantarflex the Horizon or move load line posterior	None
Softer Toe Response	Dorsiflex the Horizon or move load line anterior	None
Firmer Heel Response	Dorsiflex the Horizon or move load line anterior	Install Rounded Heel Wedge*
Softer Heel Response	Plantarflex the Horizon or move load line posterior	Remove Rounded Heel Wedge*

* To install or remove toe and heel wedges, refer to the Horizon Wedge Kit Instructions.

GAIT MATCHING® GUIDELINES

Contact College Park Technical Service if you have questions about modifying the gait match.